Datum: 28. Februar 2017

Referenz: C9826109-22022017-001

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MDRC Series


Mounting procedure:

1. Take out the protective cover from the box and place it inner side up on a clean table.

2. Remove the film from the inner side of the protective cover.

Note: Remove the film fast and smoothly at one time.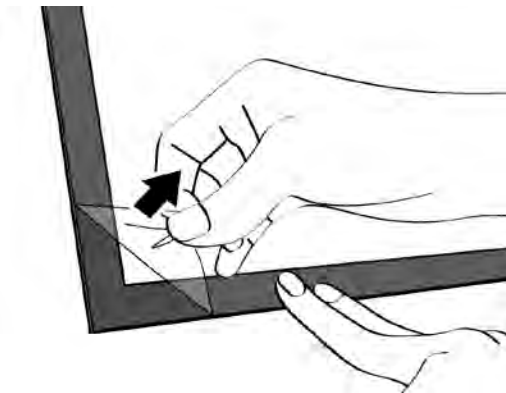


- Using gloves, align, and stick the 4 metal strips on the magnetic strips on each side of the protective cover.

Warning: The metal strips are designed for single use only and shall never be re-applied after sticking the protective cover to the front of the LCD monitor since this could cause the protective cover to fall off which might result in serious personal injury.

Note: Wear gloves to prevent fingerprint or dust on the protective cover.

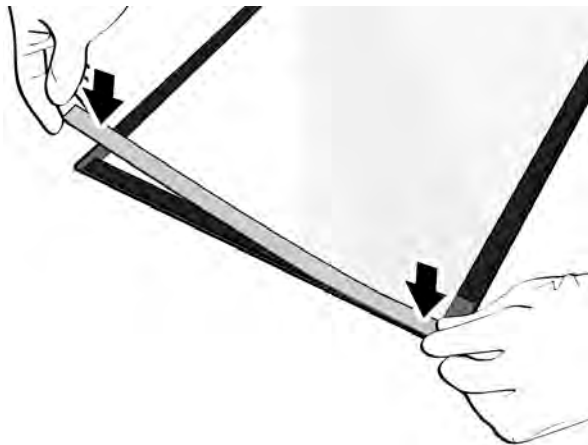4. The picture below shows the completed 4 metal strips.

5. Tear off the double tape on the metal strips.

6. The picture below shows the tear off tape of the 4 metal strips.


7. Well align and stick the protective cover on the front cover of the LCD monitor from bottom to top.


Warning: Make sure that protective cover is properly positioned on the LCD monitor.

Warning: The metal strips are designed for single use only and shall never be re-applied after sticking the protective cover to the front of the LCD monitor since this could cause the protective cover to fall off which might result in serious personal injury.

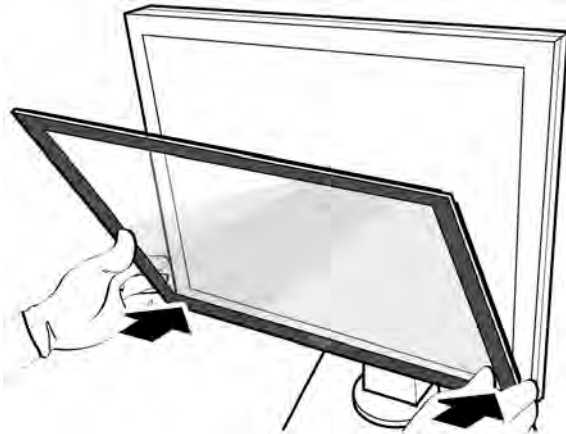
8. After mounting, press around the rim of the protective cover and leave it for at least 4 hours to allow the tape to adhere properly.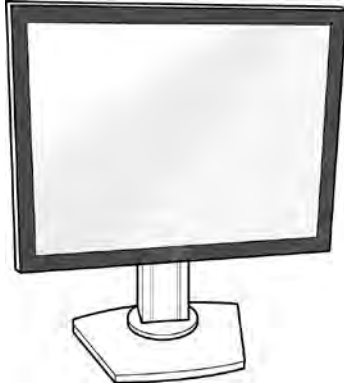

9. Remove the protective film from the outer side of the protective cover. The mounting procedure is then completed.

Note: Remove the film fast and smoothly at one time.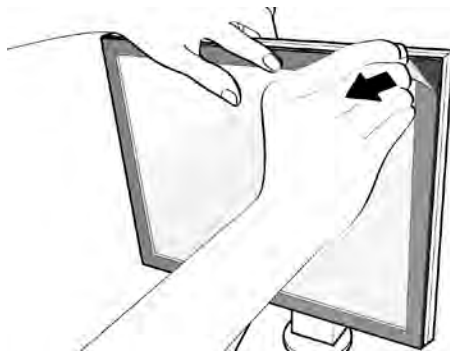

Dismounting procedure:


1. Take out the suction cup from the box and stick it on the upper-right corner of the protective cover.

2. Use the suction cup to remove the protective cover from the LCD monitor.

Warning: The metal strips are designed for single use only and shall never be re-applied after sticking the protective cover to the front of the LCD monitor since this could cause the protective cover to fall off which might result in serious personal injury.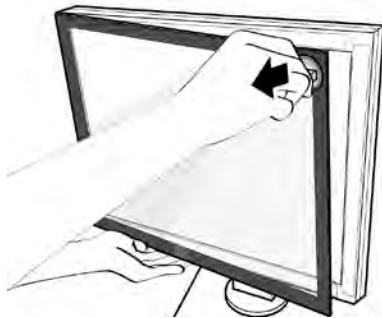

3. After removing the protective cover from the LCD monitor, push the tab to remove the suction cup from the protective cover. The dismantling procedure is then completed.