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




intens

WITTENSTEIN intens GmbH · Walter-Wittenstein-Straße 1 · 97999 Igersheim · Germany

Contact:  
Phone:  
Fax:  
e-mail:



Date: 07.12.2017

## User information regarding poller screw placement

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
Dear FITBONE® User,

WITTENSTEIN intens would like to point out that all poller screws must be put in place with the dummy; in situ placement together with the original FITBONE® is allowed in exceptional cases only. Please ensure that the FITBONE® passes the poller screws easily and without resistance during insertion. Even minor damage may result in increased notch stress and, consequently, implant breakage. If steering the FITBONE® past the poller screws proves difficult, we recommend to rework the medullary cavity with a respective medullary reamer.

If you have any further questions, please do not hesitate to contact us.

Yours sincerely,

**WITTENSTEIN intens GmbH**



### WITTENSTEIN intens GmbH

Walter-Wittenstein-Straße 1  
97999 Igersheim · Germany  
Tel. +49 7931 493-0  
Fax +49 7931 493-10906  
info@wittenstein-intens.de  
www.wittenstein-intens.de

Geschäftsführer  
Roman Stauch  
Dr. Gregor Strasser  
AG Ulm HRB 680692  
USt-IdNr.: DE 198016360

Sparkasse Tauberfranken  
SWIFT/BIC: SOLADES1TBB  
IBAN EUR: DE63 6735 2565 0000 0670 58

Deutsche Bank AG Würzburg  
SWIFT/BIC: DEUTDEM790  
IBAN EUR: DE53 7907 0016 0840 8718 00