

05.01.2006
po/we

Productinformation on our carebeds

- ***Arminia II***
- ***Westfalia III***
- ***Theutonia III***
- ***Fortuna***

2001 till 31.03.2003 – year of manufacture

Dear Sirs,

As you already know, BURMEIER carebeds complies with the appropriate standards and the other regulations of the surveillance authority. This is of course applicable to the above mentioned BURMEIER carebeds.

In addition to that we would like to tell you for precautionary reasons that individual cases have resulted into a snatch operation of the profile of the backrest.

To avoid any resulting disadvantages for the care needed person we would like to point out the following on the occasion of certain well known cases:

According to the allegation of standards the load capacity of the backrest is only designed up to 79 kg.

Additional loads exceeding the maximum of 79 kg are unconditionally to avoid!

If loads exceeding the maximum should be realised, you can of course order fortified constructions. Nevertheless we have since April 1st, 2003 – due to our constant product improvement – the above mentioned carebeds installed with strengthened backrests.

Would you please follow these additional product informations if the a.m. carebeds are inserted. Please also bring your manual instructions up to date by enclosing this letter.

For any questions we remain at your service.

With best regards

Burmeier GmbH & Co.
Kommanditgesellschaft

signed by company management