## Kundeninformation / Customer information

betreffend / concerning

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**Details on the affected products and recommended safety measures**

The safety and satisfaction of our customers are our topmost priorities. Although only one claim has been registered, the company Hermann Bock is carrying out the present preventive measure.

The circuit board of the described double drives is equipped with a diode, which disconnects the emergency lowering battery from the parallel connected power supply. In individual cases it could come to a defect of this diode, and therefore to a damage of the emergency lowering battery. In case of a failure of the diode, the supply voltage meets the battery, which after a short time can lead to a damage of the battery and to heat development in that very area.

The double drives type MD200 are subjected to a 100% control after the production. Based on our findings a damage of these diodes is very unlikely.

Nevertheless, Hermann Bock GmbH recommends as an additional measure of safety to disconnect the emergency lowering battery for the backrest. In order to do this, you pull off the battery from the battery case and disconnect the contact plugs:

The emergency lowering of the backrest is without the battery not electronically possible but it can be carried out mechanically.

The mechanic emergency lowering is described in an additional information to the existing user's manuals (page 15 a). The additional information will be available as an annexe to this customer information.

In case, that the electronic emergency lowering is of essential meaning for the use environment, respectively for the particular case of application, a gratis retrofit kit can be offered as a corrective measure from mid-October 2019 (calendar week 42).

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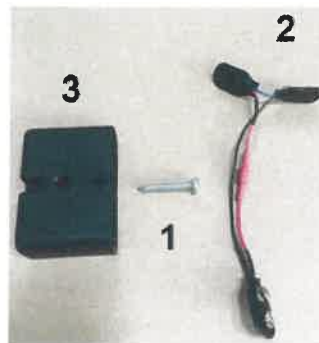
The retrofit can be done by the customer. Kindly check before the retrofit, if the article code and manufacturing date of your drives comply with the below mentioned indications of the effected drives! If one of the indications does not comply, do not carry out a retrofitting.

The emergency battery of the retrofit kit is secured with an additional diode in the connecting cable of the battery. In order to enable the retrofit of the double drive MD200 you receive the following parts:

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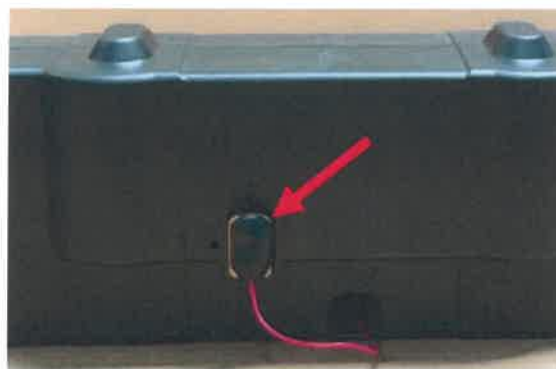
Kindly follow the below mentioned steps for the retrofit:

#### 1. Removal the old battery

To remove the old battery pull it out of the battery case of the drive and disconnect it from the connecting plugs.

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

#### 2. Assembly of the battery holder

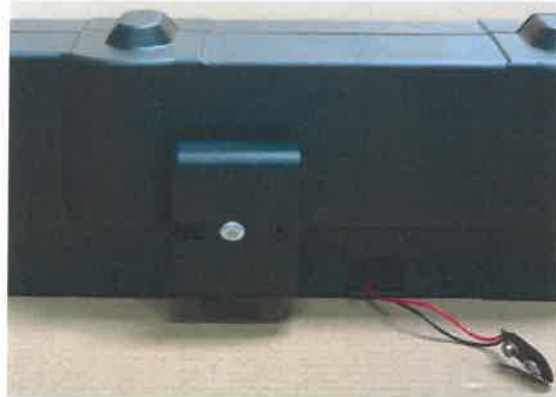To connect the new battery holder with the torx screw, please use the designated drilling next to the battery case. Fasten the screw with a hand screw driver. You would need a screw driver type „Torx T25“. Pay attention that you do not fasten the screw too tight. The casing of the holder should not deform.

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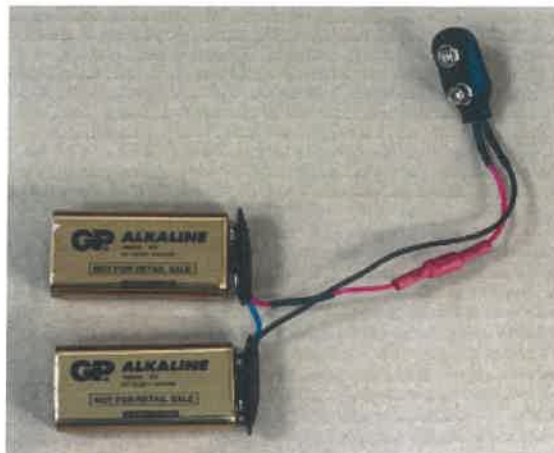
**3. Connection of the batteries**  
Connect two 9 V monobloc batteries to the contacts of the provided retrofit cable.

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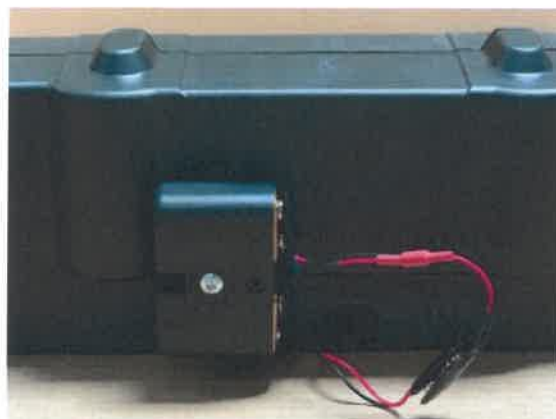Put the batteries into the fastened battery holder. Connect each cable plug of the retrofit kit with the plugs coming out of the motor.

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The motor is now successfully retrofitted.

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## Identification of the manufacturing date

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- This customer information is related to the double drive MD 200 with the article code 450792/800.00453
- The corrective measure concerns all drives that have been produced before the 5<sup>th</sup> calendar week 2013
- You recognize the manufacturing date as follows:  
Date: FCW03= manufacturing date of the drive  
E= 2012, F= 2013 CW = calendar week  
The corrective measure should be carried out for the drives with the identification marking:
  - ECW01 bis ECW52
  - FCW01 bis FCW04

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(limoss GmbH & Co. KG/ Her-