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




Sword Xiantao Disposable Protective Products Factory



Urgent Field Safety Notice

Commercial Name of the affected product: Medical Face mask

FSCA-identifier: N/A

Type of Action: Recall the batch of the goods immediately.

Date: April 12.2021

Attention: The reference number assigned by the manufacturer is SIM-2021-282-001.

Details on Affected Devices:

On December 8, 2020, our company(**Sword Xiantao Disposable Protective Products Factory**) sold a total of 10 million of medical face masks to Germany by air through the intermediary **FE protective GmbH**, and FE protective GmbH sold them to the authorities in Bavaria, Germany. The batch number of these goods was **20201110**, the production date was **2020-11-10**, and the validity period was 2025-11-09. The details as following:

Invoice No.: 20SIM282-001A/B/C/D/E/F

Medical Face Mask with ear loop 3ply TYPE IIR w/o webbing round elastic

Blue

(50pcs/bag/box, 2000pcs/ctn)

Lot No.: 20201110

QTY: 10,000,000pcs

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
Description of the problem:

Sword Xiantao Disposable Protective Products Factory received an informed that the medical face masks failed to pass the spot test. Data from the authorities' sampling tests showed that some of the respirators' air permeability tests showed a heterogeneous image, and the pressure difference between the five test points of a single mask also showed large fluctuations. Upon receipt of this complaint, Sword Xiantao Disposable Protective Products Factory contact the middleman FE protective GmbH and inform him to stop all sales of the goods immediately.

This batch of goods with a total amount of 10 million had been sold to the Bavarian authority. On March 3, FE informed that the Bavarian authority will give the masks out to an external labor to test once more. On March 26, 2021, FE informed that the results of the external laboratory tests still failed. Therefore, Sword Xiantao Disposable Protective Products Factory finally decided to recall this batch of goods immediately.



Sword Xiantao Disposable Protective Products Factory



Take Measures: Contact to arrange the shipment and Recall the batch of the goods immediately.

Contact Reference Person:

FE protective GmbH
Bahnhofstr. 31, 94032 Passau, Germany
Call: +49 162 1000500
Contact: Michael Bogner
Mail: michael@protective-sports.com

EC-Rep:

Shanghai International Holding Corp. GmbH(Europe)
Eiffestrasse 80, 20537 Hamburg, Germany
Call: +49-40-2513175
Contact: Liang Jin
Mail: shholding@hotmail.com



Sword Xiantao Disposable Protective Products Factory

