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


Customer address



23.04.2020 TH



**Safety notice - Additional product information about the screwdriver T6 Linos REF 26-975-39-07**

Dear Sirs,

The purpose of this safety notice is to inform you about an additional product information initiated by Gebrüder Martin GmbH & Co.KG when using the T6 Linos screwdriver.

**Description of circumstance**

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
In the application, excessive mechanical torsional stress can cause the screwdriver to break at the working end.

Below you will find a corresponding illustration of such a breakage at the working end.



**Possible risks of breakage**

The breakage and the fragment stuck in the screw head could lead to permanent damage of the tissue. It may not be possible to explant the plate.



**NOTE for correct application of the T6 Linos screwdriver REF 26-975-39-07**

If you cannot use an alternative product to the one mentioned above, we recommend performing the insertion of the multidirectional smartDrive® locking screws only until you feel a noticeable locking effect. It is not necessary to continue the rotation movement after this resistance; in worst case, this could result in damage of the screwdriver. The rotary motion itself should only come from the hand, rotation from the upper arm should be strictly avoided.

In addition to the product listed here, the Safety Notice must be considered for the following additional articles:

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
Material	Description German	Description English
26-975-33-07	smartDrive Schraubendreher kurz T5 drehb	smartDrive Screwdriver T5 rotating short
26-975-36-07	smartDrive Schraubendreher kurz T6 drehb	smartDrive Screwdriver T6 rotating short
26-975-37-07	smartdrive Schraubendreher T6	smartDrive Screwdriver T6
26-975-38-07	smartDrive Schraubendreher kurz T5	smartDrive Screwdriver T5 short
26-975-39-07	smartDrive Schraubendreher kurz T6	smartDrive Screwdriver T6 short
26-975-40-07	Schraubendreherklinge smartDrive T6, L	Screwdriver blade smartDrive T6, L

**Transmission of safety notice "Additional product information"**

Please ensure that all users of the product mentioned and other affected people in your organisation who need to be informed are aware of this safety notice for additional product information. If you have submitted the products to third parties, please forward a copy of this safety notice "Additional product information" or inform the contact persons below. Please keep this information at least until you have informed all the people in your company who has to be informed.

**Gebrüder Martin GmbH & Co.KG**



Your responsible health authority for medical devices has received a copy of this safety notice "Additional product information about the screwdriver T6 Linos REF 26-975-39-07".





Please do not hesitate to contact us in case of any further questions.

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Kind regards,



**Safety notice – Additional product information about the screwdriver T6 Linos REF 26-975-39-07**

**Annex**

**Reply-Fax to +49 7461 706-350  
to Gebrüder Martin GmbH & Co.KG**

Customer \_\_\_\_\_

Customer-No. \_\_\_\_\_

**NOTE for correct application of the T6 Linos screwdriver REF 26-975-39-07**

If you cannot use an alternative product to the one mentioned above, we recommend performing the insertion of the multidirectional smartDrive® locking screws only until you feel a noticeable locking effect. It is not necessary to continue the rotation movement after this resistance; in worst case, this could result in damage of the instrument. The rotary motion itself should only come from the hand, rotation from the upper arm should be strictly avoided.

In addition to the product listed here, the Safety Notice must be considered for the following additional articles:

Material	Description German	Description English
26-975-33-07	smartDrive Schraubendreher kurz T5 drehb	smartDrive Screwdriver T5 rotating short
26-975-36-07	smartDrive Schraubendreher kurz T6 drehb	smartDrive Screwdriver T6 rotating short
26-975-37-07	smartdrive Schraubendreher T6	smartDrive Screwdriver T6
26-975-38-07	smartDrive Schraubendreher kurz T5	smartDrive Screwdriver T5 short
26-975-39-07	smartDrive Schraubendreher kurz T6	smartDrive Screwdriver T6 short
26-975-40-07	Schraubendreherklinge smartDrive T6, L	Screwdriver blade smartDrive T6, L

**We herewith confirm:**

Receipt of the letter safety note "Additional product information about the screwdriver T6 Linos REF 26-975-39-07" from Gebrüder Martin GmbH & Co.KG from April 2020.

\_\_\_\_\_  
Place, Date

\_\_\_\_\_  
Name / Position of signatory

\_\_\_\_\_  
Organisation / Stamp / Signature of signatory

