

Field Safety Notice 20-090 Cordring art no 9607S

Dear valued customer,

Wing Plast has reviewed and updated the Instruction for Use for the Cordring.

Please note the following changes:

**Picture 1, Warning: *"Do not stretch the cordring more than necessary. This may cause the cordring to break."***

**Picture 5, Warning: *" Do not twist the cordring as shown here. This may cause the cordring to break."***

**Warning: *Check the umbilical cord the first 24 hours after.***

Please make to sure to comunicate the changes throughout your organisation.

If you are a distributor, please make sure to correspond the changes to all customers using the Cordring.

Wing Plast kindly ask you to fill out the Customer Reply form and send it to the mailadress on the Customer Reply form.

Smålandsstenar 2021-05-12

Lena Nurbo  
Head of Quality Assurance / Regulatory Affairs

Wing Plast AB  
Nitgatan 11, SE- 333 33 Smålandsstenar  
Sweden  
Tel: +46 371 330 30  
Fax: +46 371 330 31