

December 2018

Urgent FIELD SAFETY NOTICE

IV Pole Collar for Trima Accel® Automated Blood Collection System

Type of action: Device Upgrade

Dear Valued Customer,

Terumo BCT is dedicated to providing you with the highest quality support and remains committed to communicating information regarding Terumo BCT products. This letter provides information about the new IV pole collar accessory for the Trima Accel system.

New Trima Accel systems will be shipped with an IV pole collar (shown in Figure 1 below). This feature has been added to avoid a pinch point between the IV pole cross bar and the machine while lowering the IV pole. On new machines, the service technicians will attach the IV pole collar to the IV pole during installation.

If the operator does not hold onto the IV pole while pressing the IV pole release button, the IV pole may drop in a free fall. If the operator's hand or any other body part is in the path of the free-falling IV pole, the operator could be injured. Because this is an added safety feature, do not remove the collars after they are installed on the IV pole.



Figure 1: Photo of the IV pole collar installed on the Trima Accel IV pole.

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


Figure 2: Photo of the IV pole collar prior to installation.

Your local Terumo BCT representatives will work with you to ensure IV pole collars are provided and installed on Trima Accel systems already in the field. On new machines, service technicians will attach the IV pole collar to the IV pole during installation.

We appreciate your continued support and look forward to serving you. If you have any questions, please contact your Terumo BCT representative or your regional customer support center. Please note that your National Competent Authority has been notified about this Field Safety Notice.

Sincerely,



Terumo BCT, Inc.
 10811 West Collins Ave.
 Lakewood, Colorado 80215-4440
 USA
 USA Phone: 1.877.339.4228
 Phone: +1.303.231.4357
 Fax: +1.303.542.5215

Terumo BCT Europe N.V.
 Europe, Middle East and Africa
 Ikaroslaan 41
 1930 Zaventem
 Belgium
 Phone: +32.2.715.0590
 Fax: +32.2.721.0770

Terumo BCT Asia Pte. Ltd.
 89 Science Park Drive
 #04-25 (Lobby B)
 The Rutherford
 Singapore 118261
 Phone: +65.6715.3778
 Fax: +65.6774.1419

Terumo BCT Latin America S.A.
 La Pampa 1517-12th Floor
 C1428DZE
 Buenos Aires
 Argentina
 Phone: +54.11.5530.5200
 Fax: +54.11.5530.5201

Terumo BCT Japan, Inc.
 Tokyo Opera City Tower 49F,
 3-20-2, Nishi-Shinjuku,
 Shinjuku-ku, Tokyo 163-1450,
 Japan
 Phone: +81.3.6743.7890
 Fax: +81.3.6743.9800