

Quality Control VIDAS (QCV) - REF : 30706

CUSTOMER NOTIFICATION

LOTS NUMBER	EXPIRATION DATE	LOTS NUMBER	EXPIRATION DATE
788735201	2005/05/05	792075801	2005/09/09
789677401	2005/06/09	792450701	2006/03/01
790295401	2005/06/23	792656801	2006/03/07
790313001	2005/07/01	793596601	2006/03/07
790754001	2005/08/05	793609501	2005/09/09
791708501	2005/08/25		

KC2 – FCA N° 2005 - 04 – Reagent

To the attention of the laboratory Responsible

Marcy l'Etoile, May 3rd 2005

**QUALITY CONTROL VIDAS
REFERENCE : 30706**

LOTS : 788735201	Exp. Date : 2005/05/05
789677401	2005/06/09
790295401	2005/06/23
790313001	2005/07/01
790754001	2005/08/05
791708501	2005/08/25
792075801	2005/09/09
792450701	2006/03/01
792656801	2006/03/07
793596601	2006/03/07
793609501	2005/09/09

Madam, Sir,

You are a user of Quality Control VIDAS (Ref. 30706) of one of the following lots :

788735201, 789677401, 790295401, 790313001, 790754001, 791708501, 792075801, 792450701, 792656801, 793609501 and 793596601.

We have registered some remarks that the TV1 ratio value was found to be too low whereas the VIDAS / mini VIDAS system did not have any problem of pipetting or optical.

As a result of these observations, an investigations was conducted by our quality control laboratory.

A new target value for the TV1 ratio has been redefined for these lots.

TV1 ≥ 5.6 instead of ≥ 6.3

For your next controls with these lots please check your QCV results taking into account the TV1 value mentioned above. (TV1 ≥ **5.6**)

A label indicating the TV1 value to be used will be inserted in the kit of the next affected lots. Our technical department is at your disposal for any other information.

Tel. XXXXXXXX – Fax. XXXXXXXX

Best regards,

Customer Service