
ACCU-CHEK[®] *Active*

(EN) Important note:

- ▶ Touch the orange application area with your finger when applying your drop of blood to the test strip.

If the value measured is extremely low and is different to what you feel it should be, or if the screen shows Lo or Err, please repeat the test with a new test strip. Please ensure that the drop of blood you use is large enough.

If you have any questions, you should contact your local customer services.

ACCU-CHEK[®] *Active*

ACCU-CHEK[®] *Plus*

(DE) Wichtiger Hinweis:

- ▶ Berühren Sie das orange Auftragsfeld mit dem Finger, wenn Sie den Blutstropfen auf den Teststreifen auftragen.

Falls der Messwert extrem niedrig ist und nicht Ihrem Empfinden entspricht, oder Lo oder Err angezeigt wird, wiederholen Sie die Messung mit einem neuen Teststreifen. Stellen Sie sicher, dass der Blutstropfen groß genug ist.

Bei Fragen wenden Sie sich bitte an Ihren lokalen Kundendienst.

Roche Diagnostics GmbH

Sandhofer Strasse 116
D-68305 Mannheim
Telefon (06 21) 7 59-0
Telefax (06 21) 7 59-28 90

Registergericht Mannheim
HRB 3962
Aufsichtsrat:
Dr. Franz B. Humer, Vorsitzender

Geschäftsführung:
Dr. Jürgen Schwiezer, Vorsitzender
Dr. Manfred Baier, Staffan EK,
Dr. Volker Pfahlert,
Burkhard G. Piper,
Dr. Susanne Raehs,
Peter-Claus Schiller,
Prof. Dr. Dr. Klaus Strein