

Anlage 2


Dade Behring Marburg GmbH
P. O. Box 11 49
35001 Marburg, Germany
www.dadebehring.com

Phone: +49 (6421) 39-5405
Fax: +49 (6421) 39-3070

DADE BEHRING

To whom it may concern

Unser Zeichen
K. U. Nabeck/BR-29805.doc

 Durchwahl
+496421-39-5405

Fax
+496421-39-3070

Datum
November 30, 2005

**Field Correction
on
BEP® 2000 / BEP® 2000 Advance System**

Dear Valued Dade Behring Customer,

Our records indicate that you are using a BEP® 2000 / BEP® 2000 Advance System. This is to inform you that BEP® 2000 / BEP® 2000 Advance System software version 1.23.2 and all previous versions contain a software error that might lead to a repeated pipetting of the same pre-diluted sample into one pre-dilution well. Thereby, it may occur that liquid from overflowing wells contaminates adjacent wells within the dilution plate.

The problem is caused by different amounts of diluted sample volume required by the respective assays. In case of using only original Dade Behring assay definitions the problem can only occur if Enzygnost Anti-EBV/IgM II or Enzygnost Anti-EBV/IgA II is run in a mixed batch mode (different assays on the same plate frame) together with other Enzygnost IgM parameters.

Therefore, please be advised not to run Enzygnost Anti-EBV/IgM II and Enzygnost Anti-EBV/IgA II in a mixed batch mode. If you are using non-Dade Behring assay definitions please make sure that you do not run assays with different amounts of diluted sample volume in a mixed batch mode.

Please note that user defined modifications are not supported by Dade Behring as they may affect performance of the system and assay results.

Dade Behring has initiated measures for a timely correction of the software.


If you have any questions or need assistance please contact your local Dade Behring representative.

Please distribute this information to all appropriate personnel in your laboratory, retain a copy in your files, and forward this information to all parties that may use this product, including others to whom you have transferred this product.


DADE BEHRING

We apologize for the inconveniences this action causes and thank you for your continued support of Dade Behring Infectious Disease Diagnostics products.

Sincerely,

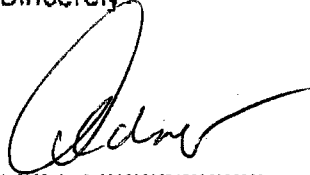

.....
Dr. Norbert Dedner
Director
Field Quality Assurance

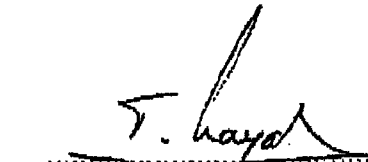

.....
Tarek El Nayal
Director
Global Marketing IDD