

Dade Behring Marburg GmbH
P. O. Box 11 49
35001 Marburg, Germany
www.dadebehring.com

Phone: +49 (6421) 39-3070
Fax: +49 (6421) 39-3070

DADE BEHRING

To whom it may concern

Our Ref.
BR-03107.doc

Ext.
+496421-39-3070

Fax
+496421-39-3070

Date
2007-04-18

Field Correction

**BEP® 2000 / BEP® 2000 Advance System
Software Version 1.23.3 and Previous Versions**

Dear valued Dade Behring Customer,

Our records indicate that you are using a BEP® 2000 / BEP 2000 *Advance* System. Dade Behring was informed by its supplier that after ignoring several hardware error messages an incorrect assignment of results can occur.

It was observed that after pressing the IGNORE button several times after plate transport error messages appeared, the instrument read again the first plate of the worklist instead of the second plate. The obtained results were erroneously assigned to the second plate.

Please be advised not to use the IGNORE button in case of a mechanical plate transport error message to avoid erroneously assigned results.


We will update the Technical Documentation in order to provide more clarity.

If you have any questions or need assistance please contact your local Dade Behring representative.

Please distribute this information to all appropriate personnel in your laboratory, retain a copy in your files, and forward this information to all parties that may use this product, including others to whom you have transferred this product.

We apologize for the inconveniences this action causes and thank you for your continued support of Dade Behring Infectious Disease Diagnostics products.

Sincerely

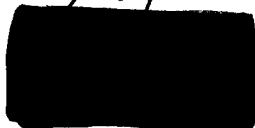

Director
Quality Systems & Compliance



Director
Global Marketing PP/IDD