

APPENDIX 1

To the attention of the laboratory manager

Address

City, Date

IMPORTANT :
Product Information
VIDAS C. *difficile* Toxin A&B ref. 30118
lot 814609601 - exp. 06/03/2008

Dear Sir, Dear Madam,

We are happy to count you amongst our users of VIDAS C. *difficile* A&B (CDAB) kits ref. 30118 lot 814609601.

The yellow insert in this kit mentions that the dilution of the samples, standard, and controls must be 1:16 in the sample diluent. This indication is erroneous. The dilution must be 1:6 as mentioned in the technical sheet (200 µl samples in 1000 µl R1 sample diluent).

The overdilution of samples CDAB may lead to a sensitivity default of the test. In that case and according to the patients' clinical history, we recommend you to retest the samples, if need be.

We apologize for the inconvenience this may have caused and remain at your disposal should you need further information.

Best regards,

Your Customer Service