

Urgent Field Safety Notice

1003 / 1103

September 2009

IMMULITE[®] / IMMULITE 1000 Platforms

IMMULITE / IMMULITE 1000 Total T4 LKT4 Kit Lot 351–Barcode Error

Our records indicate that as an IMMULITE[®] / IMMULITE 1000 Total T4 customer you may have received IMMULITE / IMMULITE 1000 Total T4 LKT41 Kit Lot 351 or LKT45 kit lot 351. Due to an error in the barcodes provided on the kit box, the reagent wedge cannot be recognized by the instrument. As a result, you will be unable to use this kit.

To resolve this problem, the correct barcodes are attached to this notice in alphanumeric and pictorial form. Please ensure that you delete the old LKT4 kit lot 351 from the system prior to typing or scanning in the replacement LKT4 lot 351L barcodes.

Kits can be deleted from the database in the Kit Entry/Kit Information window and using the F5 key:

IMPORTANT: Use the F5 key with care. Pressing F5 deletes any window that is displayed, not only kit information. This may cause loss of data and/or require database repair.

IMMULITE

- Select **Data Entry**.
- Select **Kit Entry**.
- Press the **Page Down** key to locate kit lot 351. When kit lot 351 is displayed, press the **F5** key.

IMMULITE 1000

- Select **Kits**.
- Use the **Search** button to locate kit lot 351 in the Kit Information window. When kit lot 351 is displayed, press the **F5** key.

NOTE: The Kit Entry or Kit Information window will not update automatically. You can verify that the information was deleted by accessing another window and then returning to Kit Entry or Kit Information window.

Please forward this letter to whomever you may have distributed this product.

We apologize for the inconvenience that this situation has caused. Thank you for your patience and continued support.

If you have any difficulty, please contact your local Siemens Technical Solutions Center.

Siemens Healthcare Diagnostics Inc.

Glyn Rhonwy
Llanberis, Gwenedd LL55 4EL
United Kingdom

+44-1286-871-871
www.siemens.com/diagnostics

Page 1 of 4


100 Test Kit

T4

IMMULITE®/IMMULITE® 1000 Systems

REF LKT41

LOT 351L

 2010-07

↓ **Adjustor**



ZCEK9D3AZAAAS2G

#1



TGEA98B842AC

#2



WIAMXIAIASIP

#3



3LAND6AITB6M


500 Test Kit

T4

IMMULITE®/IMMULITE®1000 Systems

REF LKT45

LOT 351L

 2010-07

↓ **Adjustor**



ZCEK9D3AZAAAS2G

#1



TGEA98B842AC

#2



WIAMXIAIASIP

#3



3LAND6AITB6M

Page intentionally blank.

Siemens Healthcare Diagnostics Inc.

Glyn Rhonwy
Llanberis, Gwynedd LL55 4EL
United Kingdom

+44-1286-871-871
www.siemens.com/diagnostics

Page 4 of 4