



Urgent Field Safety Notice
Product Correction
Immediate Action Required

Date Issued August 12, 2013

Product **Phosphorus Reagent**

- List Number 7D71-22; Lots 64736UN12, 03991UN13, and 51611UN13
- List Number 7D71-31; Lots 74805UN12 and 03992UN13

Explanation The Phosphorus reagent lot numbers listed above have been labeled with incorrect expiration dating.

Patient Impact Patient results could potentially be impacted if the Phosphorus reagent is used after the corrected expiration date (see below).

Necessary Actions Update existing inventory and manually track the impacted lots with the **corrected** expiration dates shown in the table below:

Product Name	List Number	Lot Number	INCORRECT Expiration Date on Product Labeling	CORRECT Expiration Date
Phosphorus Reagent	7D71-22	64736UN12	14 JAN 2014	14 SEP 2013
		03991UN13	01 APR 2014	01 DEC 2013
		51611UN13	13 JUL 2014	13 MAR 2014
	7D71-31	74805UN12	14 JAN 2014	14 SEP 2013
		03992UN13	01 APR 2014	01 DEC 2013

Please retain a copy of this letter for your laboratory records. If you have forwarded any of the lots above to another laboratory, please provide a copy of this letter to them.

Contact Information We sincerely regret any inconvenience this may cause your laboratory. If you or any of the health care providers you serve have any questions regarding this information, U.S. Customers please contact Customer Service at 1-877-4ABBOTT. Customers outside the U.S., please contact your local area Customer Service.