



Vidal MMI Germany GmbH
Monzastraße 4
63225 Langen – Germany
Phone +49 (0) 6103 2076-0

Urgent Safety Information / Product Notification

Manufacturer: Vidal MMI Germany GmbH

Subject: Log4j Security Vulnerabilities (CVE-2021-44832 and CVE-2021-45105)

Affected products: OntoDrug AMTS

Product code / UDI: OntoDrug AMTS 4.2.5

Date of notification: 06-Jan-2022

Required action: Apply Patch version 4.2.6

Author: Vanessa Freund, Verantwortliche Person (PRRC)

Short summary

Vidal MMI Germany GmbH has become aware of a potential security issue.

The security issue is caused by the Log4j logging library, versions 2.16.0 and 2.17.0. OntoDrug AMTS version 4.2.5 uses Log4j version 2.16.0.

The following measures are to be taken by the customer/user

The security problem (CVE-2021-44832 and CVE-2021-45105) must be fixed immediately by applying the provided patch version 4.2.6 on the FTP server of Vidal MMI Germany.

Corrective actions by Vidal MMI Germany GmbH:

1. All customers will receive this product notification to be informed about the possible security vulnerability.
2. Vidal MMI Germany GmbH will provide a software update on the FTP server accessible for the customers, which will permanently fix the cause.

Please inform all affected staff in your department about the content of this product notification and store this information until all actions have been completed at your site.

We regret any inconvenience and thank you in advance for your cooperation.

Please contact our support team if you need further informations (Monday to Thursday 9am-4pm and Friday 09am-3pm, excluding public holidays).

E-Mail: support@mmi.de