Innsbruck, 09.06.2009

[REDACTED]

[REDACTED]

[REDACTED]

# LEONHARD LANG

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Leonhard Lang GmbH, Archenweg 56  
6020 Innsbruck, Austria  
www.LeonhardLang.at

Tel. +43-512-33 42 57, Fax 39 22 10  
Medical@LeonhardLang.at

Dear Customer and User of our Defibrillation electrodes DF20, DF20N, DF20NC, DF31, DF31L.

Upon the implementation of new measuring equipment we have identified a potential risk associated with the connectors of our defibrillation electrodes DF20, DF20N, DF20NC, DF31, DF31L. All these defibrillation electrodes are compatible with Physio-Control defibrillators.

It was found that some of the connectors do not slip easily into the sockets of the defibrillator or an extension cord of the defibrillator. In such cases increased force or some wiggling is required to insert the connector fully. Although we have inspected the connectors for their fit with original Physio-Control equipment, we cannot rule out that some connectors, which do not insert easily into the sockets, have slipped through our inspection. This is because all original equipment is subject to wear, which can lead to a connector passing this test, which it might not have, had a brandnew piece of original equipment been used.

We see a potential risk for patients, if (a) such a connector (b) is used with a nearly brandnew defibrillator or extension cord (c) in a situation imposing extensive stress on a user.

We therefore have decided to call back all lot numbers of DF20 DF20N, DF20NC, DF31 and DF31L starting with lot number 80505-0778.

Please, immediately stop using or distributing any of the above defibrillation electrodes of these lots and quarantine them. Please report all quantities of these products you have in stock using the attached form as soon as possible. We will in turn arrange for these products to be returned to us and will replace them shortly.

We are extremely sorry to cause this inconvenience and effort.

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**LEONHARD LANG GmbH**