

# WelchAllyn<sup>®</sup>

Quality Assurance & Regulatory Affairs  
8500 SW Creekside Place  
Beaverton, OR 97008

Dear Valued Customer:

We are writing to inform you that the directions for use of your Welch Allyn AED 10 or MRL JumpStart defibrillator contained in the user manual have been updated to improve the clarity of describing what should be done when the device displays a low battery status icon.

The primary update to the directions for use is addition of the following information regarding the low battery status.

- The flashing low battery status indicator means that the battery is beginning to weaken and should be replaced at the first opportunity. The AED 10 remains operable when the flashing low battery is activated, and it can be used on a patient in this condition or continued in use when connected to a patient if the low battery indicator activates. Replace the battery as soon as possible.
- If the AED 10 is subjected to cold temperatures near or outside of its low temperature operating limit of 32° F (0° C), the low battery status indicator may be triggered even with a new battery. The low battery indicator caused by cold temperature may cease when the device is warmed. Discharging the device to defibrillate a patient may cause sufficient warming to eliminate the low battery indicator or cause it to be intermittent.

Enclosed with this letter is a disc that contains the updated user manual for your device. Please discard the original user manual and replace it with this updated version.

We appreciate your cooperation in this matter.

Respectfully,

  
Regulatory Manager  
Welch Allyn, Inc.