

To our customers of the Dräger
Oxylog 2000 *plus* and Oxylog 3000 *plus*

February 2014

Important Safety Notice!!!

Dräger Oxylog 2000 *plus*, part number 5705080
Dräger Oxylog 3000 *plus*, part number 5704833

Error message *Poti unplugged*

Dear Madam/Sir,

Within the framework of our quality checks during production, we have noticed increasing numbers of devices displaying the error message *Poti unplugged*.

Our investigation has revealed that a limited batch of control panel PCBs – onto which the potentiometers for setting the ventilation parameters are soldered – is faulty. This can result in the contact to the respective potentiometer being interrupted. The device then discontinues ventilation, gives an audible alarm and displays the error message *Poti unplugged*. Despite our extensive quality control testing, part of the batch in question was shipped in September 2013.

If this error occurs, the user must immediately commence ventilation using an independent ventilator (manual ventilation bag) as specified in the Instructions for Use.

It is devices of the type Oxylog 2000 *plus* (part number 5705080) and Oxylog 3000 *plus* (part number 5704833) with the serial numbers listed on page 3 of this letter that are affected. Both the part number and the serial number can be found on the typeplate to the rear of the device.

Dräger Medical GmbH
Moislinger Allee 53-55
23558 Lübeck, Deutschland
Postanschrift:
23542 Lübeck, Deutschland
Tel. +49 451 882-0
Fax +49 451 882-2080
info@draeger.com
www.draeger.com
UID-Nr. DE813745277

Bankverbindungen:
Commerzbank AG, Lübeck
Konto-Nr. 014 679 500
BLZ 230 400 22
IBAN DE95 2304 0022 0014 6795 00
SWIFT-Code COBADEFF230

Sparkasse zu Lübeck
Konto-Nr. 1 071 117
BLZ 230 501 01
IBAN DE15 2305 0101 0001 0711 17
SWIFT-Code NOLADE21SPL

Sitz der Gesellschaft: Lübeck
Handelsregister:
Amtsgericht Lübeck HRB 4358 HL

Vorsitzender des Aufsichtsrats:
Professor Dr. Nikolaus Schweickart

Geschäftsführer:
Stefan Dräger (Vorsitzender)
Dr. Herbert Fehrecke
Andreas Frahm
Gert-Hartwig Lescow
Anton Schrofner

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
Page 2 / 3

Our service organization will contact you shortly to arrange for your devices to be modified free of charge.

We apologize for any inconvenience this action may cause, but believe the measures to be essential in the interests of patient and user safety.

If you have any questions, please contact your local Dräger representative.

With kind regards



Head of Key Application Field
Respiratory Care

Appendix:
List of serial numbers affected, 1 page

Oxylog 2000 plus	
Part number: 5705080	
Serial numbers	
ASEJ-0015	ASEK-0001
ASEJ-0016	ASEK-0002
ASEJ-0019	ASEK-0003
ASEJ-0020	ASEK-0004
ASEJ-0021	ASEK-0005
ASEJ-0022	ASEK-0006
ASEJ-0023	ASEK-0007
ASEJ-0024	ASEK-0008
ASEJ-0025	ASEK-0009
ASEJ-0026	ASEK-0010
ASEJ-0027	ASEK-0011
ASEJ-0028	ASEK-0012
ASEJ-0029	ASEK-0016
ASEJ-0030	ASEK-0017
ASEJ-0031	ASEK-0018
ASEJ-0033	ASEK-0019
ASEJ-0034	ASEK-0021
ASEJ-0035	ASEK-0022
ASEJ-0036	ASEK-0023
ASEJ-0037	
ASEJ-0038	
ASEJ-0039	
ASEJ-0040	

Oxylog 3000 plus	
Part number: 5704833	
Serial numbers	
ASEJ-0107	ASEK-0002
ASEJ-0108	ASEK-0003
ASEJ-0109	ASEK-0004
ASEJ-0110	ASEK-0005
ASEJ-0111	ASEK-0006
ASEJ-0112	ASEK-0007
ASEJ-0113	ASEK-0008
ASEJ-0114	ASEK-0009
ASEJ-0115	ASEK-0010
ASEJ-0116	ASEK-0011
ASEJ-0117	ASEK-0012
ASEJ-0118	ASEK-0013
ASEJ-0119	ASEK-0014
ASEJ-0120	ASEK-0015
ASEJ-0121	ASEK-0016
ASEJ-0122	ASEK-0017
ASEJ-0123	ASEK-0018
ASEJ-0124	ASEK-0019
ASEJ-0125	ASEK-0020
ASEJ-0126	ASEK-0021
ASEJ-0127	ASEK-0022
ASEJ-0128	ASEK-0023
ASEJ-0129	ASEK-0026
ASEJ-0130	ASEK-0029
ASEJ-0131	
ASEJ-0132	
ASEJ-0133	
ASEJ-0134	
ASEJ-0135	
ASEJ-0136	
ASEJ-0137	
ASEJ-0138	
ASEJ-0139	
ASEJ-0140	
ASEJ-0140	