

**Call our
service number
+49 89 729970012**

Product information 2013-12-03EN Single breathing system ClevAir M86100-12300


Dear Sirs,

MPV MEDICAL informs you regarding potential nonconformities of the single breathing system ClevAir.

In the production period from January 2013 to September 2013 a modified process for preparing the exhalation valve line at the single breathing system was used. It is possible that surplus adhesive flowed into the control line and blocked it. In this case the exhalation valve wouldn't work correctly.

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
We would like to ask you to send above mentioned single breathing systems with the LOT number from xx.01.13 to xx.09.13 back to MPV MEDICAL. We will exchange these breathing systems free of charge.

We apologize for the inconvenience of this complaint.

We would like to ask you to fill out the attached complaint document and to return it to MPV MEDICAL by facsimile, +49 89 729970044 or by e-mail, @mpvmedical.com.

If you have more questions, don't hesitate to contact me.

Sincerely


Technical product manager
Safety officer
MPV MEDICAL GmbH

Seite 1 von 1