

January 19, 2017

URGENT RECALL – C2 CryoBalloon Controller Cap

Dear Valued Customer,

C2 Therapeutics has recently become aware of a potential product issue with the C2 CryoBalloon Ablation System that may occur during use. At this time, C2 Therapeutics is collecting information and plans to perform a recall of the affected product. C2 Therapeutics recommends that **affected product should not be used**. Additional information for this product issue, occurrence and safety, product affected, recommended actions and product disposition, and company contact information are detailed below.

Product Issue

The Controller Cap may crack as the Controller Cap is tightened onto the Controller, which could compromise the integrity of the Controller Cap and result in damage to the Controller Cap. This could result in fragmentation of the top of the Controller Cap and could potentially injure the device user.

Occurrence and Safety

There has been one occurrence of this product issue to date that was reported to C2 Therapeutics on December 20, 2016. There have been no prior injuries or adverse events associated with this product issue to date.

Identification of Affected Product

C2 Therapeutics has identified the source of this product issue as the Controller Cap. C2 Therapeutics has two versions of the Controller Cap, and the issue occurs with only one of the Controller Cap versions. The Controller Caps that may be affected by this product issue are easily visually identifiable prior to use (the Controller Caps have a small hole in the center of the top of the Controller Cap). At this time, all C2 CryoBalloon Controller product lot numbers that were distributed between June 23, 2016 and December 20, 2016 may have Controller Caps that are affected by this product issue.

Product Disposition and Response Form

C2 Therapeutics recommends that products with affected Controller Caps should not be used. If you do not feel you can identify the affected Controller Cap, C2 Therapeutics recommends that you **do not use** any product unless a C2 Therapeutics representative is present.

C2 Therapeutics is taking immediate action to correct this issue. Over the next three weeks, C2 Therapeutics will be mailing replacement Controller Caps for all Controllers in your inventory. C2 Therapeutics will include a labeled shipping envelope for you to return the original Controller Caps as verification that the Caps have been replaced. This action will not require you to identify affected Controller Caps or to contact us for a RMA. **Please complete and return the recall response form included with the replacement Controller Caps.** In the event, we have not provided a sufficient number of replacement Controller Caps, you may request additional Controller Caps.

If preferred, C2 Therapeutics can correct this issue with either of the methods described below. Please contact us immediately if you would rather replace the Controller Caps with either of these alternative methods. If we do not hear from you, we will be sending replacement Controller Caps as soon as we receive them from our supplier.

- OPTION 1

Company representatives are trained to identify the affected Controller Caps based on an external feature. The company representative will inspect the product in your inventory and replace the Controller Cap as necessary. Your sales representative will complete and return the recall response form.

- OPTION 2

All Controllers in your inventory may be returned to C2 Therapeutics for replacement at no charge. To return product, a returned Material Authorization (RMA) number may be requested by email or telephone at the contact information below. To expedite return of your corrected product, **please write the RMA number in large letters on the return packaging. Please complete and return the recall response form included with the replacement Controller Caps.**

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
Recall and RMA Contact Information

To request an RMA number or for questions or more information regarding this recall, please use the contact information below. The replacement Controller Caps should be available to replace all Controller Caps by the end of January.

1. To speak with someone, call Theresa Brandner at +1-650-521-5921 or your sales representative. For communication in German, please contact Medical Device Safety Service at vigilance@mdss.com or +49 511 6262 8630.
2. To request an RMA number, send an email to rma@c2therapeutics.com or call +1-650-521-5921.

This field correction is being made with the knowledge of the relevant National Competent Authorities in Europe and the U.S. Food and Drug Administration. Please forward this Field Safety Notice to all those within the organization who need to be aware of this product issue until April 30, 2017. If the product has been transferred to another organization, please provide contact information for that organization to C2 Therapeutics, Inc.

C2 Therapeutics regrets any inconvenience that may result from this action and appreciates your patience as we resolve this Controller Cap product issue. Please be assured that maintaining patient safety and quality are our utmost priority.



C2 Therapeutics, Inc.

FIELD CORRECTION RESPONSE FORM

Acknowledgement and Receipt Form

Response is Required

«CUSTOMER_NAME»

«STREET_ADDRESS»

«CITY», «STATE» «ZIP_CODE» «COUNTRY»

CUSTOMER NUMBER: «CUSTOMER_NUMBERS»

REF.: MDD17-009

Replacement Controller Caps for C2 CryoBalloon Controllers

- I have read and understand the instructions provided in the customer notification letter
- The affected Controller Caps have been replaced and are being returned to C2 Therapeutics
- Relevant instructions have been communicated/distributed to all required facility personnel

Contact Information	
Name	
Title	
Hospital	
Telephone	
Fax Number	
Email address	

Signature of Receipt and Acknowledgement	Date

Upon completion of and signing the form, please return the form by any of the following methods:

- Faxing this completed form to C2 Therapeutics Recall Coordinator at +1-650-556-1145 (alternate +1-650-521-5921)
- Sending a PDF copy of the completed form to rma@c2therapeutics.com

If you have any questions regarding this action, please feel free to contact your C2 Therapeutics Sales Representative or Theresa Brandner-Allen at +1-650-521-5921 (8:30am – 5:00 pm PST, Monday through Friday).

For your reference below is a list of the affected products our records show your facility has purchased.

Model Number	Quantity		Model Number	Quantity		Model Number	Quantity

Field Safety Corrective Action C2 Therapeutics, Inc.

Lot Number	Expiration Date	Quantity	Lot Number	Expiration Date	Quantity
06242016-01	1-Jan-2017	10	11282016-03	2-Jun-2017	10
06292016-02	6-Jan-2017	10	09202016-01	21-Mar-2017	5
07062016-01	8-Jan-2017	10	09222016-01	23-Mar-2017	1
07082016-02	13-Jan-2017	10	09232016-01	28-Mar-2017	10
07082016-03	14-Jan-2017	10	09232016-02	27-Mar-2017	8
07152016-01	21-Jan-2017	10	09292016-01	4-Apr-2017	10
07152016-02	20-Jan-2017	9	09302016-03	4-Apr-2017	2
07212016-01	25-Jan-2017	10	10052016-02	19-Apr-2017	6
07252016-01	28-Jan-2017	10	10142016-01	18-Apr-2017	2
07262016-01	9-Feb-2017	10	10192016-01	24-Apr-2017	10
07282016-02	2-Feb-2017	5	10212016-01	25-Apr-2017	10
08022016-01	17-Feb-2017	10	10242016-01	26-Apr-2017	10
08032016-02	5-Feb-2017	5	10262016-02	1-May-2017	10
08112016-01	16-Feb-2017	9	10282016-02	8-May-2017	10
08232016-01	26-Feb-2017	10	11022016-01	9-May-2017	10
08262016-01	2-Mar-2017	10	11042016-01	10-May-2017	10
09022016-01	8-Mar-2017	10	11092016-01	15-May-2017	10
09092016-01	14-Mar-2017	10	11142016-02	16-May-2017	6
09122016-02	15-Mar-2017	4	11152016-02	18-May-2017	10
09192016-01	19-Mar-2017	5	11162016-01	21-May-2017	8
09192016-02	21-Mar-2017	10	11282016-02	30-May-2017	7