

## **URGENT MEDICAL DEVICE FIELD CORRECTION**

September 7, 2021

**ATTN: MATERIALS MANAGEMENT OR QUALITY DEPARTMENT**

**RE: CMAX 3, CMAX 3-DRIVE, CMAX 3E, and CMAX 3E-DRIVE Surgical Tables**

Dear Valued STERIS Customer:

STERIS is voluntarily implementing a field correction for all CMAX 3, CMAX 3-DRIVE, CMAX 3E, and CMAX 3E-DRIVE Surgical Tables manufactured between November 24, 2020 – July 5, 2021. Our records indicate that your facility purchased one or more of the Surgical Tables included in this field correction. The complete list of affected serial numbers is included as attachment to this letter.

**Description of the product** – The CMAX 3, CMAX 3-DRIVE, CMAX 3E, and CMAX 3E-DRIVE Surgical Tables are intended to support and position patients during surgical procedures carried out in an operating room, and during transfer to and from the patient's bed. CMAX surgical tables shall be operated in a controlled environment by a healthcare professional familiar with appropriate techniques with such equipment.

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
**Description of the problem** – STERIS has identified that the circuit board within the auxiliary (override) control may have been incorrectly manufactured, and over time, may cause the circuit board to short, potentially resulting in undesired table movement or the table becoming unresponsive to user.

**STERIS Action** – A STERIS Service Technician will arrange an onsite inspection of your table(s) for replacement of the auxiliary control.

**User Action** – While the risk of malfunction is low, the table should be both visually inspected and functionally tested prior to each use. Should an issue with table functionality be observed, the table should be removed from service until proper repairs can be conducted.

We apologize for any inconvenience this matter may cause, and as always, STERIS is dedicated to supporting our products and valued Customers. If you have questions regarding this matter, please contact STERIS Customer Service at +33 (0) 5 56 93 94 94 or your local STERIS Representative.

Sincerely,



List of Affected CMAX 3 and CMAX 3-DRIVE CMAX 3E, and  
CMAX 3E-DRIVE Surgical Tables by Serial Number

178736	182280	183828	185296
178829	182281	183829	185572
179111	182354	183830	185573
179684	183016	184389	185574
179685	183018	184411	185576
179686	183019	184412	185645
179687	183020	184413	185646
179688	183021	184414	185647
179689	183022	184415	185805
179691	183023	184416	186350
181458	183183	184744	186351
182186	183184	185276	186353
182276	183185	185293	186649
182277	183818	185294	186650
182278	183827	185295	188456
182279			