

Important Information

February 2007

For MÖLLER-WEDEL Operating Microscopes VM 900


P/N 656 109, 656 110, 656 111, and 656 116

We received notice that a microscope VM 900 slipped out of the receptacle since the turning knob was not fastened. To avoid reoccurrence and potential injury to the patient, we request from all users of the VM 900:

1. Check all microscopes VM 900 in your possession and ensure that the turning knob is properly tightened. Instruct the OR personnel to check the tight mounting of the microscope every time the microscope is used.
2. Advise the OR personnel to remain familiar with the content of the operating manual of this microscope, in particular with the statement:

Attention: Prior to each startup, check for secure engagement of all connecting points on the equipment.

3. Have the enclosed label attached to the microscope below the turning knob as shown in the following picture:

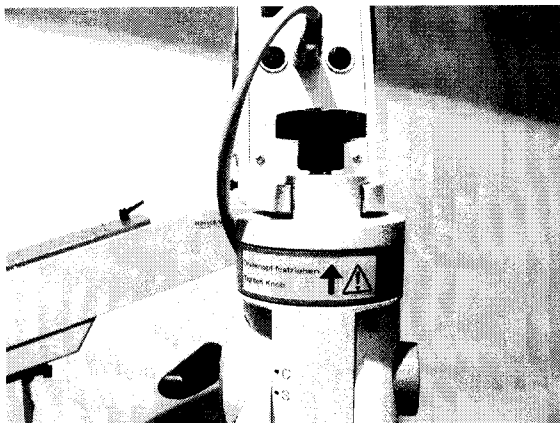

Drehknopf festziehen

Tighten Knob



Label 'Tighten Knob'

The triangular symbol means 'Read the operating manual'

This action is regulated by law. Therefore we need your confirmation that you took the above described actions and report this confirmation back to us with the part number and serial number of your microscope(s). Please use the attached confirmation letter and fax or mail it to Möller-Wedel's address.

Thank you for your cooperation.

MÖLLER-WEDEL GmbH

Encl.: Confirmation letter

Confirmation Letter

Please complete this letter and send or fax it to:

MÖLLER-WEDEL GmbH
Rosengarten 10
D-22880 Wedel
Germany

Fax: +49 4103 709 355

Name and address of the hospital or establishment:

Möller microscope VM 900 in your establishment:

P/N	Serial Number
_____	_____
_____	_____
_____	_____

I herewith confirm that the following actions were performed according to the IMPORTANT INFORMATION letter issued by Möller-Wedel in February 2007:

1. We checked all microscopes VM 900 in our possession for tight mounting and instructed the operating room personnel to check the proper mounting every time before use.
Yes No
2. We advised the operating room personnel to observe the operating manual.
Yes No
3. We attached the label 'Tighten Knob' to the microscope as instructed.
Yes No

Date: _____

Name and position of signee: _____

Signature: _____