FB-805-1  
Kundeninformation/  
Customer information

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Revision: 01

**An:** Dermatologen  
**To:** Dermatologists

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**Background:** *The starting point was the output of a negative and illogical score in the analysis with the DermaFC. The internal self-test was performed before the measurement. In the user training and in the user manual, only one (one- or two-digit) score value or 0 is evaluable, so that the negative score value must be regarded as an error message. The error message is not evaluated as an adverse event because no patient or user was harmed or could have been harmed. As part of the internal development evaluation, it was found that there is the possibility that the medical devices evaluates too few valid spectra and extrapolated the data to the examined surface.*

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*The device is not ready for further operation and should therefore not be used.*

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*Please confirm the receipt of this customer information.*

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*Read this customer information carefully.*

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*A Magnosco service employee will contact you.*

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*Please assist the Magnosco service employee in obtaining the pseudonymized patient data.*

*Magnosco checks patients' measurements for the validity of the number of spectra.*

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*If it turns out that one or more patients are affected, the customer service will contact you.*

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*If it turns out that one or more patients are affected, please inform the affected patient and invite him / her to a follow-up appointment.*

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*The costs of a necessary re-examination due to the nonconformity will be covered by Magnosco.*

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*If you have any further questions or concerns after checking this notification, please call our customer service on Monday through Friday from 7:00 am to 7:00 pm CET on (+49)030-912075370. Outside office hours, callers are asked to leave a voice message or, in the event of emergencies, to the on-duty staff. You can also send your inquiries by e-mail to [service@magnosco.de](mailto:service@magnosco.de).*

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*This recall will be reported to the BfArM and, as appropriate, to other competent authorities, notified bodies and regulatory authorities.*

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*We thank you in advance for your cooperation and regret any inconvenience caused by this recall.*

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Magnosco GmbH