



Address

21st July 2005

Children Seat Invacare® Tornado und Invacare® Typhoon

Dear Invacare® Dealer,

with our invoice no. you received the power wheelchair.....

In order to facilitate the maintenance of these power wheelchairs, you are able to pull up the seat in case of a defective battery or stroke motor, and by this means you have free access to the part in need of repair.

However, even persons not accurately instructed in helping the patient, could grasp the chair at the seat and then pull it up. At the same time the stroke motor could come out of the guidance what is perhaps not immediately remarked due to the bellow serving as protection against possible squeezing risks.

In order to avoid this, we recommend to add an additional metal plate.

Of course, the seat still remains adjustable in width and depth.

You will receive the additional plate as well as an assembly drawing from which you can take exact instructions how to fit this plate.

In case of further questions we will be at your disposal.

Kind regards

INVACARE Deutschland GmbH

Ralf Bükler
Manager Quality

Ilona Wiemann
Product Manager