

Siemens AG, MED AX PLM-CW R&F, P.O. Box 12 66, 91294 Forchheim

### Customer Safety Advisory Notice

Name	Marianne Popp
Department	MED AX PLM-CW R&F
Telephone	+49 (9191) 18-9432
Fax	+49 (9191) 18-9911
Mobile	+49 175 2226629
E-mail	marianne.popp@siemens.com
Your letter of Our reference Date	mp June 12, 2006

### Possible patient hazard due to unintended system movement

Dear Valued Siemens Customer:

This letter is intended to remind you of the importance of following the safety warnings for your system to ensure the most safe and effective use of the product.

System movements can be activated via the remote control console in the control room. We kindly remind you that objects should not be placed on or near the vicinity of this remote control console. These objects may shift and accidentally activate the system movement, thus representing a potential risk to both the patient and operator.

In addition, please remember that patients positioned under the x-ray tube for bedside exams should always be supervised by trained personnel. If this is not possible, we strongly recommend moving the bed outside the systems collision range.

Please inform your staff and maintenance personnel accordingly and ensure that this advisory is inserted into your user manual.

We appreciate your cooperation with this advisory.

Sincerely,

Siemens Aktiengesellschaft



Marianne Popp  
Product Manager



Siegfried Schneider  
Quality Manager

#### Siemens AG

Group: Medical Solutions  
Group Executive Management: Erich R. Reinhardt, Group President;  
Hermann Requardt, Siegfried Russwurm, Klaus Stegemann

Postal address:  
Siemens AG  
MED AX PLM-CW R&F  
P.O. Box 12 66  
91294 Forchheim

Office address:  
Siemensstr 1  
91301 Forchheim  
Tel: +49 (9191) 18-0  
Fax: +49 (9191) 18-9999

Siemens Aktiengesellschaft: Chairman of the Supervisory Board: Heinrich v. Pierer; Managing Board: Klaus Kleinfeld, Chairman, President and Chief Executive Officer; Johannes Feldmayer, Thomas Ganswindt, Joe Kaeser, Edward G. Krubasik, Rudi Lamprecht, Eduardo Montes, Jürgen Radomski, Erich R. Reinhardt, Hermann Requardt, Uriel J. Sharef, Claus Weyrich, Klaus Wucherer  
Registered offices: Berlin and Munich, Commercial registries: Berlin Charlottenburg, HRB 12300, Munich, HRB 6684  
WEEE-Reg.-Nr. DE 97061698, WEEE-Reg.-Nr. DE 23691322