



X-Ray Products
1678 South Pioneer Road
Salt Lake City, UT 84104-4298
USA
tel 801 973 5000
fax 801 973 5050

FIELD SAFETY NOTICE AND CORRECTION

Dear Varian Customer:

Varian has determined the PaxScan 4336R power cable used with the enclosure, may develop a short when the cable is damaged due to misuse, wheeling over, excessive bending/twisting or mistreatment. In the event of cable damage and resulting cable short, there is a risk of excessive heat in the damaged area, possibly radiating down the cable from the damage point.

Because of this, we have provided the enclosed Warning Labels for you to forward to end users of this panel with enclosures, to attach to cables. We also request that you attach to enclosure cables that you have in stock. Varian will make replacement cables incorporating a 6 inch strain relief available on a free-of-charge exchange if requested.

Varian advises all Customers and end users to avoid mistreatment, excessive bending, wheeling over or misuse of the cable, to inspect the cable regularly, and replace the cable when any damaging event occurs or when any damage to the cable is known or visible

Sincerely,

