

Siemens AG, H CP XP MK, Allee am Röthelheimpark 2, 91052 Erlangen

<To the person in charge of the unit where the SIEMENS product is operated, and the administrative head of organization>

Contact person of  
the Regional Unit  
Department

Telephone  
Fax  
E-Mail

Date

### Safety Advisory Notice

To all users of the Acquisition Workstation in combination with SIEMENS Mammomat Inspiration mammography system

Contact person of  
the Business Unit  
Department

H CP XP MK WH IN  
+49 9131

Telephone  
Fax  
E-Mail

+49 9131  
@siemens.com

Date

### **Acquisition Workstation table in combination with Mammomat Inspiration/Fusion mammography system: Potential issue if the table is not mounted on the floor**

Dear customer,

This letter is to inform you of a potential issue and risk of injury when using the Mammomat Inspiration Acquisition Workstation table (Operator table w. radiation shield material no. 10549892 when it is not fixed to the floor.

#### **When does this malfunction occur and what are the potential risks?**

If the monitors are positioned too far from the table top, the Acquisition Workstation table can become unstable with a risk of it falling over and causing injury to the user.

#### **What steps can the user take to avoid the potential risk of this issue?**

We strongly recommended positioning the monitors over the table top if the table is not fixed to the floor in order to avoid falling over.

Siemens AG  
Healthcare Sector; Leitung: Hermann Requardt  
Clinical Products Division; Leitung: Britta Fuenfstueck  
X-Ray Products; Leitung: Andre Hartung

Hartmannstr. 16  
91052 Erlangen  
Deutschland

Tel.: +49 (9131) 84 0

Siemens Aktiengesellschaft: Chairman of the Supervisory Board: Gerhard Cromme;  
Managing Board: Joe Kaeser, Chairman, President and Chief Executive Officer; Roland Busch, Lisa Davis, Klaus Helmrich,  
Hermann Requardt, Siegfried Russwurm, Ralf P. Thomas  
Registered offices: Berlin and Munich, Germany; Commercial registries: Berlin Charlottenburg, HRB 12300, Munich, HRB 6684  
WEEE-Reg.-No. DE 23691322

### How will the issue finally be resolved?

Siemens is preparing a field modification to resolve this potential issue (XP060/14/S). You will be informed by your local Siemens service representative as soon as this is available. The modification includes a new base plate that provides improved stability.

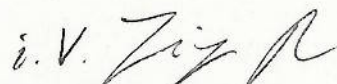
We appreciate your understanding and cooperation with this safety advisory notice and ask you to immediately instruct your personnel accordingly. Please ensure that this safety advisory notice is placed in the system's instructions for use. Your personnel should remain aware of this issue until the modification is implemented.

If you have sold or otherwise disposed of this equipment and it is no longer under your control, we kindly ask that you forward this Safety Advisory Notice to the new user of the equipment. Please also inform us about the new owner of the equipment.

We apologize for any inconvenience this may cause.

Sincerely Yours

  
Ralf Gerharz  
CFO H CP XP

  
Jürgen Buckow  
H CP XP QM

### Acknowledgement of receipt

Customer address:

---

---

---

I hereby confirm as the owner / responsible operator of the <  > with the Serial number \_\_\_\_\_ (optional) that I received the following document:

### Safety Advisory Notice

<  > with <  >

Place, Date \_\_\_\_\_

Name \_\_\_\_\_

Signature \_\_\_\_\_