

Date: 8th August 2019

Dear Customer,

Advisory Notice

Affected Products: REMOTE-CONTROLLED R/F SYSTEM FLEXAVISION

Thank you for using FLEXAVISION.

As described in this manual, follow the instructions below. Do not apply any liquid to the device.

Do NOT use the equipment at places where the liquid may enter.

The equipment is not designed to be waterproof.

Invasion of any liquid should cause electric shock, system failure or malfunction.

Do NOT spill any liquid, such as contrast medium, saline, or disinfectant, onto the equipment.

In case that such liquid drip on equipment surfaces, wipe it off immediately.

Any such liquid entering into system electronics may cause failure or malfunction.

Should liquid drip on the operation panel or enter the covers, immediately turn off the power and contact your Shimadzu service representative.

«Remarks»

As a precaution, we will ask the distributor and the Field service engineer to install the seal gasket and visually inspect the control unit on routine maintenance. If there are indications of corrosion, we will ask the distributor to replace the control unit as preventive actions.